



FLOOR

mm  
MANACA  
MEDICAL


PAD75100, PAD75180 AND PAD90200

## GEKO ANTISKID

Anti-slip mat that collects blood, irrigation fluids, urine, water and other liquids present in surgeries.



**GEKO**  
Antiskid

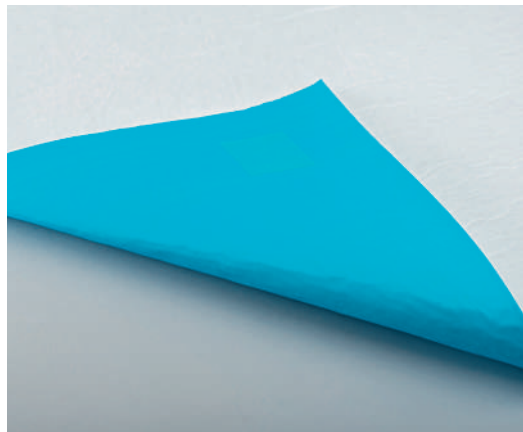

### FEATURES

- ✓ Composed of 2 layers: A first layer of white absorbent airlaid non-woven fabric and a second layer of blue non-slip polyethylene with a matte PE film.
- ✓ With 4 double-sided adhesives that leave no residue and are easily removed.
- ✓ Does not slip in wet or dry conditions.
- ✓ Absorbent from 2 l/m<sup>3</sup> to 4 l/m<sup>3</sup>.
- ✓ Prevents falls in the surgical environment.
- ✓ Reduces cleaning time and resources.
- ✓ Quick and easy to install.
- ✓ Simple and economical final disposal.
- ✓ Disposable, single-use product.





## FLOOR

PAD75100, PAD75180 AND PAD90200

### GEKO ANTISKID

REF.	SIZE	ABSORBENCY m <sup>2</sup>	TOTAL ABSORBENCY	WEIGHT UNIT
PAD75100	75X100 cm	2.0 liters / m <sup>2</sup>	1.50 liters	150 grams
PAD75180	75X180 cm	2.0 liters / m <sup>2</sup>	2.70 liters	260 grams
PAD90200	90X200 cm	2.0 liters / m <sup>2</sup>	3.60 liters	380 grams

### PRESENTATION

REF.	BOX	UNITS/BOX	WEIGHT BOX
PAD75100	101x77X13 cm	75	14 kg
PAD75180	77x92X25 cm	50	19 kg
PAD90200	92x52X15 cm	40	17 kg

### COMPLIANCE

Non-sterile Class I medical device. According to Regulation (EU) 2017/745 . Latex free.

### USE AND INDICATIONS FOR USE

- Disposable, single-use mats.
- Operating room. Surgical sinks and sterilization centers.
- Absorbent mat to collect fluids. Non-slip matte blue polyethylene in contact with the ground, absorbent white airlaid facing up.
- After placement, it can be walked on or carts pass by. The product can be cut.
- The mat does not retain fluid, it needs to be folded before lifting to limit fluid leakage when discarding.



SINGLE USE



LATEX FREE



NON STERILE